PRODUCT NAME: IFU XAILIN WASH 20x5 ml EN FR IT DE NL SALES	LANGUAGES: EN FR IT DE NL	SIZE: 297x210 mm
MATERIAL CODE: AW-00717_1	REVISION LEVEL: 1	ARTWORKER: Barbara Nuvole
COLOUR #1:  Black 100%	COLOUR #4:	PROGRAMME: ADOBE CC
COLOUR #2:	COLOUR #5:	FONTS: Myriad Pro SemiCondensed
COLOUR #3:	COLOUR #6:	
FINAL APPROVALS		
COUNTRY MANAGER #1:	COUNTRY MANAGER #2:	REGULATORY:
COUNTRY MANAGER #3:	COUNTRY MANAGER #4:	
NOTES:		

EN Instruction for use

Xailin® Wash is a solution presented in single dosage format without preservatives, which acts by means of mechanical drag forces, and, therefore, is especially indicated for subjects with sensitive eyes and nostrils. The main functions of the **Xailin® Wash** are:

- With contact lenses:
- it prevents the dehydration of hydrophilic contact lenses.
- it rinses all types of contact lenses.
- it is suitable for the dissolution of enzyme tablets.
- Ocular instillation:
- it reduces the discomfort resulting from adverse environmental conditions, such as settings with air conditioning/heating, dry environments, smoky environments, etc.
- Nasal irrigation:
- Symptomatic relief of nasal catarrh (rhinitis).

INSTRUCTIONS FOR USE

Separate an ampoule of the product and open it with freshly washed hands, twisting the top part and ensuring that the ampoule does not come into contact with any surface.

- **Rinsing of contact lenses and dissolution of enzyme tablets:**
- Rinsing: After completing the cleaning process, or whenever necessary, place a contact lens on the palm of your hand and rinse with a generous stream of saline solution, rubbing the lens with your fingertips.
- Dissolution of enzyme tablets. Use the solution according to the manufacturer's instructions for the use of the enzyme tablets.
- **Direct use on the eye surface:**
- Either with or without the contact lens placed within the eye, exert mild pressure on the ampoule in order to release a few drops of **Xailin® Wash** over the eye surface as many times as needed.
- Washing of the eyelids: Soak a sterile gauze with **Xailin® Wash** by exerting pressure on the ampoule, and apply the soaked gauze over the eyelid.
- **Direct use within the nostrils**
- Tilt your head backwards.
- Insert the nozzle within either nostril and exert mild pressure on the single-dose ampoule. Repeat the same process within the opposite nostril. Straighten your head in order to enable the release of any mucus plugs and clean the excess product.
- Do not blow your nose forcefully within the 10 minutes following the administration. In the case of infants, administer the product by exerting minimal pressure on the ampoule in order to avoid any risk of contamination of the middle ear.

PRECAUTIONS

- Keep out of reach of children.
- This product has no preservatives. After opening the ampoule use immediately and discard after use to avoid risk of contamination and being any cause of infection.
- One ampoule must only be used by the same person.
- The same ampoule must not be used to treat eyes and nostrils at the same time.
- Do not use the product after the expiry date.
- Do not use if the ampoule is damaged.
- Consult a specialist in case of eye or nostril irritation.
- Do not store at temperatures above 25°C.

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